

Unitron a/s



PiggyTrim

▪ modern hoof care

A safe tool to assist systematic hoof care and hoof trimming of sows.

PiggyTrim operates by letting the sow to go into the box, after which she is hoisted up lying on her stomach with the legs on either side of the longitudinal tubes. The sow calms down when the floor disappears beneath it, and you can work undisturbed with the hoof care.

EASY OPERATION:

The box is operated by two buttons and there is no safety paddles or similar to keep in mind.

WHERE TO PLACE THE PIGGYTRIM:

Placed PiggyTrim convenient in relation to internal logistics.

POSSIBLE FOR INDIVIDUAL CUSTOMIZATION:

As desired, bolt foot or embedding bracket can be supplied. Shall PiggyTrim be placed on a slatted floor, it must be supported at the top, typically by bolting it to a wall or to the rafter structure. Once PiggyTrim is fully raised, you can pass unhindered under it and therefore not in the way on the walking area.

TREATMENT:

The hoof care is done with a grinder with a flap disc attached and a hoof scissors. The treatment takes 5 - 10 minutes per sow.



- Gently for the sow
- Good working height
- High ground clearance
 - can placed in walking areas
- Easy to use
- Only one pillar - gives easy access to the sow



ACCESSORIES

Angle grinder for hoof care

- Max. disc diameter: 125 mm
- Voltage: 220V 240V
- Nominal power consumption: 1100W 40W
- Output power: 640W
- No load rotational speed range: 0-2800 r/min 0-10500 r/min
- Torque: 3 nm
- Spindle thread diameter: M14
- Vibration level: 6 m/s²
- Length: 365 mm
- Height: 113 mm
- Cable length: 4 m
- Weight: 2,1 kg

Item:
1116-7105



Safety glasses Crackerjack

Clear and anti-fog coating.
Close fit which provides optimal protection.
EN 166 1 FT, EN 170.

Item.: 1312-5070



Dust mask FFP2 with valve 10 pcs.

Comfort: Cup shape.
Exhalation valve for lighter breathing resistance.
Adjustable nose clip for optimized fit.

Item: 1311-2355



Metal gloves for hoof care

Can be used on both hands.
Size Medium: Item.: 1116-7301
Size XL: Item.: 1116-7302



Piggy Tar Spray 400 ml 12 pcs. packages

Tar has been a well-known and good hoof product for many years, as the natural antiseptic properties of tar are used.

Item.: 1116-1803



Piggy Pro bandage 9 cm x 6 meters

Blue. 12 pcs. packages.
Item.: 1116-1681



Piggy Salicylic Care Paste 300 ml

Piggy Salicylic Care Paste contains 38% salicylic acid and is used for hoof trimming as disinfection under cotton wool and bandage. Medium thick gel that sticks easily to the hoof and cotton wool.


Item.: 1223-0233



Piggy hoof disc True grit

- The disc are coated with heavy stone
- Quick pruning and long life
- Can be used on wet and dry hooves
- No dust or smell
- Suitable for one-handed angle grinder
- Diameter 115 mm with 22.23 mm hole

Item: 1116-1207

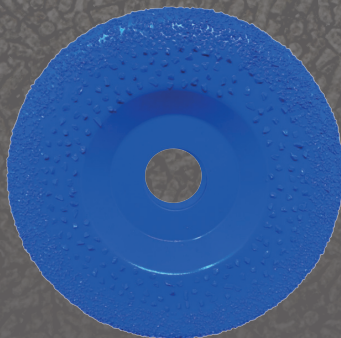

Piggy Lanolin Spray 250 ml 12 pcs. packages

A natural product based on lanolin which makes it extremely adhesive and thus very protective for outside in a stable environment.

Item.: 1116-1804



Piggy Care Stater Kit for 50 sows

Treatment package for 50 x wounds on legs and hooves

- Bandages
- Dry Pads
- Salicylic Care Paste
- Lanolin Spray
- Pistol

Item.: 1116-1692

